
## SIS - Shoulder Immobiliser Sling

**EN** SHOULDER ORTHOSIS  
**DE** SCHULTERORTHESE  
**FR** ORTHÈSE DE L'ÉPAULE  
**ES** ORTESIS DE HOMBRO  
**IT** ORTESI PER SPALLA  
**NL** SCHOUDERORTHOSE  
**DK** SKULDERTORTOSE  
**FI** OLKANIVELORTOOSI  
**SE** AXELORTOS  
**NO** ORTOSE, SKULDER  
**PL** ORTEZA STAWU BARKOWEGO  
**GR** ΝΑΡΘΗΚΑΣ ΩΜΟΥ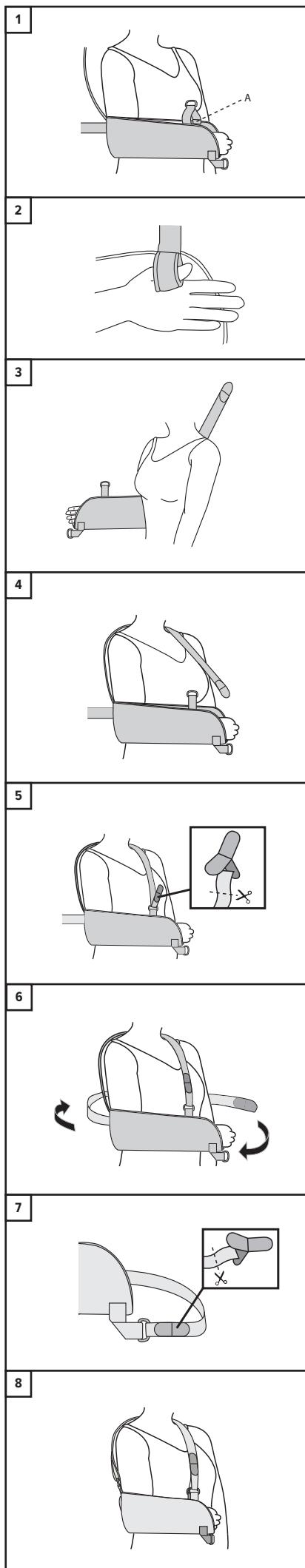

### ENGLISH

#### Product Description

The Shoulder Immobiliser can be used as an arm sling or as a shoulder immobiliser and it is designed to be used on both left and right arms.

#### Usage Information (See diagram)

- Place arm in sling and ensure the elbow is held securely in place. Fix Hook / Loop at position 'A' shown in diagram.
- Thumb loop provided if required.
- Take the shoulder strap up across the back.
- Take shoulder strap over opposite shoulder. Position neck pad as appropriate.
- Pass shoulder strap through plastic loop and fix at required length. Trim strap if required and re-position hook/loop Y-Tab.
- Pass immobiliser strap around body.
- Pass through plastic loop, adjust and fix to required position. Trim strap if required and re-position hook/loop Y-Tab.
- Check patient is comfortable and sling is offering required support and immobilisation.

#### Indications for Use

- Can be used for an array of injuries and conditions where the restriction of shoulder and / or elbow movement is required such as:
- Dislocation.
  - Strains and sprains.
  - Pre and post operatively.
  - Rehab stages.
  - Other injuries.
  - Brachial plexus or peripheral nerve.

#### Contraindications

- Do not apply the product in direct contact with broken skin.

#### Warnings and Precautions

We recommend the initial fitting of this device be conducted by a suitably qualified healthcare professional who will advise the period of use.

- Carefully read all instructions and warnings prior to use.
- Follow all instructions to ensure proper performance of the device.
- Do not use if liniments, ointments, gels, creams or any other substances have been applied to the affected area.
- Do not re-use for another patient, doing so risks cross infection and can compromise product integrity.
- Perform regular skin and circulation checks, especially patients with diabetes, vascular deficiencies and neurological conditions.
- Should any adverse reactions occur, please cease use and contact your healthcare professional or provider of this product.

#### Washing and Care Instructions

- Close over all hook/loop straps.
- Hand wash sling in cool water using mild detergent.
- Rinse and dry flat.
- Do not spin or tumble dry.
- Do not use fabric conditioner at any time.
- Check for any degradation that may affect use.
- Periodically check hook/loop fasteners are operating securely.

#### Storage and Transport Conditions

Store in a cool, dry place out of direct sunlight, in the original packaging.

#### Recycling and Disposal

Packaging and constituent parts should be recycled or safely disposed in accordance with local or national laws.

#### Serious Incident

Report any serious incident to the manufacturer and the competent authority of the EU Member state, or country in which you reside.

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