


TRIAL LUMBAR WRAP



General information about Elements Body can be found on our website.

The Trial Lumbar Wrap is an evaluation tool to establish if the user will benefit from an Elements Body compression garment. The Trial Lumbar Wrap can be used for up to two weeks for evaluation purposes. It is advised that this product is administered to no more than 10 patients for evaluation, and it can be used for a maximum of two weeks or less per user. When issuing the lumbar wrap, it is advised that the treating clinician instructs on the correct fitting to both the user and care giver.

Size	Hip
XXS	50-60 cm
XS	60-75 cm
S	75-90 cm
M	90-105 cm
L	105-125 cm

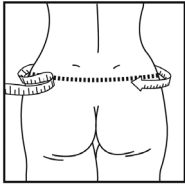

*If unsure of size, take
note of hip circumference.*

Refers to product with Serial Number _____

Trial case	Delivery date	Patient ID	Return date	Washed and inspected (date & signature)
#1				
#2				
#3				
#4				
#5				
#6				
#7				
#8				
#9				
#10				

TRIAL LUMBAR WRAP

elements
body

Product description

Textile lumbar wrap with hook and loop adjustment in centre front and with additional straps to provide support and alignment of the hips.

Intended use and Indications

The lumbar wrap is intended to be used for assessment purposes only, prior to the provision of Elements Body garment. The lumbar wrap is intended for short term use for a maximum of two weeks and is a useful assessment tool within the clinic setting.

Contraindications

The lumbar wrap is intended to be used for assessment purposes only, prior to the provision of an Elements Body garment.

Sizes

3 adult sizes (S-M-L)

3 paediatric sizes (XXS-XS-S)

Limitations

The Lumbar Wrap can be used by up to 10 patients for evaluation, for a maximum of two weeks usage per patient.

The prescriber is responsible for providing a clean and damage free product to the next user. See product for wash and care information. The product chart can be utilised to document usage. The product may only be used as an Elements Body evaluation tool.

Cautions


- Should not be used damp
- Not flame retardant
- Check garment and skin daily before usage
- Product should be washed and examined between change of users

Material Specifications

Polyester, Polyamide and Spandex. POM stays

Washing Instructions

Machine wash at 60°C between test patients.

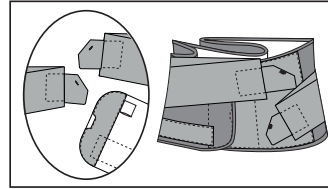

Close the hook and loop adjustment straps before washing.

Inspection

- Product is intact and free from damages.
- Seams are intact.
- Washing label is still readable.
- Hook and Loop still has good fastening ability.
- Feel the material to ensure that the stays in the back are intact.
- If the test patient is suspected of having an infection dispose of the product immediately.

Disposal

Disposal in accordance with local legislation.

Information

Any serious incident that may occur in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and / or patient is established.



Camp Scandinavia AB
Karbingatan 38
SE-254 67 Helsingborg, Sweden
Phone +46 42 25 27 01



allard

Sverige
042-25 27 01
info@camp.se
camp.se

Danmark
43 96 66 99
info@camp.dk
camp.dk

Suomi
09 350 76 30
info@camp.fi
camp.fi

Norge
23 23 31 20
info@camp.no
camp.no

UK/Ireland
42 932 8177
info@allarduk.co.uk
allarduk.co.uk

USA
888-678 6548
info@allardusa.com
allardusa.com

International
42 25 27 01
info@allardint.com
allardint.com