
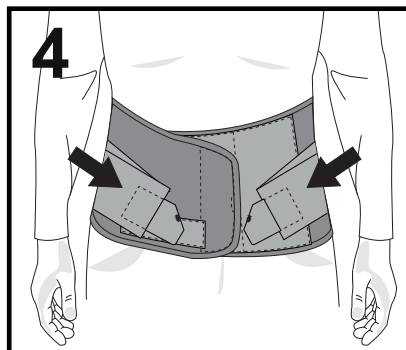
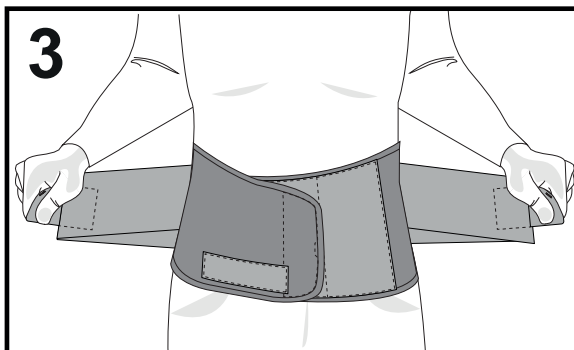
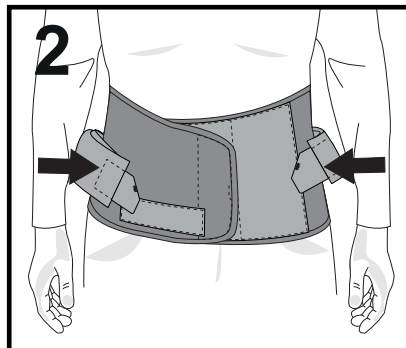
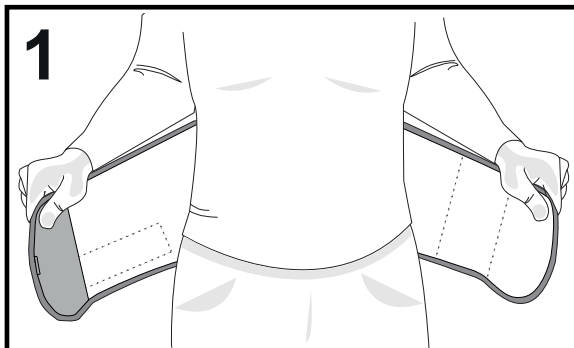
Instructions For Use

User Instructions

Trial Lumbar Wrap
Art. no. 39346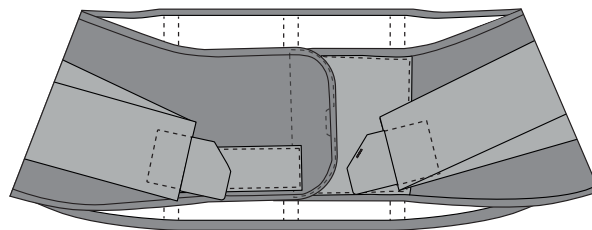

EN

Product description

Textile lumbar wrap with hook and loop adjustment in centre front and additional straps to provide support.

Intended Use & Indications

The lumbar wrap is intended to be used for assessment purposes only, prior to the provision of Elements Body garment. The lumbar wrap is intended for short term use for a maximum of two weeks. The brace must be fitted by medical personnel, and the practitioner must ensure that the healthcare provider understands how it is to be used.

Contraindications

The lumbar wrap is intended to be used for assessment purposes only, prior to a the provision of an Elements Body garment.

Limitations

To be used for part of the day only, follow the prescribers' recommendation, but should not be worn longer than 8 hours per day. Usage should be two weeks or less per user

Cautions

- Should not be used damp
- Not flame retardant
- Check garment and skin daily before usage

Information

Any serious incident that may occur in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and / or patient is established.

Material Specifications


Polyester, Polyamide and Spandex. POM stays

Sustainability information

Return the product to the prescriber at the end of the trial period.

Washing instructions

Close the hook and loop adjustment straps before washing.

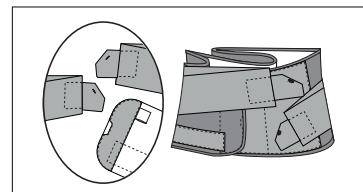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