Fitting and Adjustment Guide

Table of Contents |

English	4
German	6
Dutch	8
Spanish	10
Italian	12
French	14
Danish	16
Norwegian	18
Finnish	20
Swedish	22

Donning

1. Open the Surestep SMO from behind and underneath the foot, then pull the brace forward onto the foot. Fig. 1

2. While cupping the heel of the SMO in one hand, supinate the foot inside of the brace. Fig. 2

To do this, you will need to twist or rotate the foot outward by lifting under the ball of the foot. The ankle will also need to dorsiflex (lift up on the front of the foot). This will allow you to look down into the device under the foot and see that the heel is properly touching the back.

3. Next, push the foot back into the brace until you feel that it has made contact at the back of the heel and is positioned within the heel opening. Fig. 2

4. Maintain this position by squeezing the device while you begin to fasten and tighten the straps. Fig. 3

5. Don shoe once SMO has been properly fit.

Fig. 1

Fig. 2

Fig. 3

correct

incorrect

Fig. 4

Fig. 5

Fig. 6

Tips & Tricks

Redness

Some redness is okay! When the Surestep SMO is properly worn, it is normal for slight redness to appear in areas where the brace surrounds the foot. If blistering or severe redness occurs, please notify your healthcare professional.

Tightness

Surestep SMOs must be TIGHT to provide the compression and support your child needs to achieve stability. Remember, snug is not enough.

Growth

In order for the Surestep system to be effective, it must fit your child properly. On average, children outgrow their braces in one year. However, every child develops differently and this time frame may vary. If you begin to notice issues with the fit of your child's braces, please schedule an appointment with your healthcare professional to be evaluated.

Trim Line

Surestep SMOs should be trimmed behind the ball of the foot and to the base of the 5th (pinky) toe. Adjustments may be necessary for proper fit. Scissors are recommended for trimming device.

1. On the medial side (inside) of the foot, braces should be trimmed behind the big toe, and the ball of the foot. Fig. 4
It is important that this edge is back far enough so that it does not impinge on the boney portion of the ball of the foot.

2. On the lateral side (outside) of the foot, braces should be trimmed just behind the 5th toe. Fig. 5
Extend or push up on the 5th toe to make sure the fit is correct. This will simulate the movement the toe experiences while walking. The plastic edge of the brace should not restrict this movement.

3. If the device fits properly, there will be a gap between the edges of the plastic on the top of the foot. Fig. 6
Depending on the child, this gap may be as narrow as 3/8" on small feet (about the width of a pinky finger) or as wide as 1" on larger children (about as wide as a thumb). These plastic wings should never touch or overlap.

Care

Use mild soap and water or isopropyl alcohol to clean the plastic and velcro straps. Allow the device to air dry before donning.

Footwear

Proper footwear is required to achieve optimal correction. Always wear a wide, deep shoe with laces in combination with this product.

Surestep has developed a line of footwear that was specifically designed for children who wear orthoses. Additional width, depth and flexibility accommodate orthotic devices and make for a comfortable fit. Our shoes are available to purchase directly through your orthotist or online at surestep.net.

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