allard|<sub>N</sub><sup>T</sup>

kiddieflow™

kiddieGAIT®

kiddieROCKER®

UTPROVNINGSORTOS

SOVITUSORTOOSI

AFPRØVNINGSORTOSE

TILPASSINGSORTOSE

NOT-FOR-RESALE

[www.allardint.com](http://www.allardint.com)

## Utprovningssortos

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## NOT-FOR-RESALE

This information is only for the KiddieGAIT Not-for-Resale. These orthoses are only for temporary use to assist in appropriate product selection prior to fitting with a permanent orthosis. They serve as a valuable assessment tool to: (1) Determine appropriate size, (2) Assess gait pattern improvement, (3) Determine modifications required, and (4) Evaluate patient acceptance. If the patient feels excessive discomfort or pain, the assessment should be aborted immediately. With initial steps, some patients may feel uncomfortable due to the affect the orthosis has on gait pattern and finding position of balance. We highly recommend first assessment be done with a walking aid or in parallel bars.

The KiddieFLOW™/KiddieGAIT®/KiddieROCKER® Not-for-Resale should always have padding on the inside. The padding is machine washable and can be re-used.

The KiddieFLOW™/KiddieGAIT®/KiddieROCKER® Not-for-Resale should always be fitted with a foot orthotic on top of the footplate. This can be the child's ordinary foot orthotic, USB, SMO or other. Below you will find some questions and answers to help you get started with using the Not-For-Resale orthosis. For further information, please visit [www.allardusa.com](http://www.allardusa.com).

## Indications

KiddieGAIT® and KiddieROCKER® (all models) are intended to support drop foot, gait abnormalities following proprioceptive disorder (unstable or low tone gait), toe walking without metatarsal collapse, low tone Crouch gait in conditions such as spina bifida, cerebral palsy, muscular dystrophy and myelomeningocele. KiddieFLOW™ is intended to be used for the same diagnoses as above except for low tone crouch gait and spina bifida.

## Contraindications

KiddieFLOW™, KiddieGAIT® and KiddieROCKER® (all models) should not be used in patients with:

- Limited dorsiflexion (needing at least 5° of dorsiflexion past the neutral position)
- Very rigid structures in the foot
- Spasticity of the quadriceps
- Excessive genu valgum or genu varum misalignment
- Fixed/non-correctable pes valgus or pes varus misalignment
- KiddieFLOW™ - Crouch gait and Spina Bifida

## Limitations

Knee hyperextension (if not managed)  
(The limitations have to be addressed on an individual basis).

## Always check the following before use:

- Padding is correctly positioned on the orthosis
- Foot orthotics in place
- Both Velcros are securely fastened
- The shoe is well positioned
- Correct heel height, no hyperextension in the knee
- Never leave the patient alone when training with the Not-for-Resale orthosis

## General guidelines for foot orthotic interventions:

*Mild pronation* = use a Foot Orthotic that is deep and firm  
*Moderate pronation* = UCB (Deep Foot Orthotic with trim line below ankle).

*Severe pronation and/or tight achilles tendon* = SMO (Supra Malleolus Orthosis) or Molded Inner Boot.

## Questions & Answers

### How do I choose correct size?

Start with child's foot length and start with the KiddieFLOW™/KiddieGAIT®/KiddieROCKER® that is closest to that length. The orthosis increases in rigidity as the size increases. It is important to select final size based on weight, function, and activity level. If you determine the size you have selected is too rigid, select the next smaller size. The footplate should always be covered with an insole or foot orthotics to prevent irritation if the footplate is smaller than the foot. Many patients will require a custom insole, wedging, posting, or heel lifts to address issues of supination, pronation, valgus, varus, or to change knee extension moment.

### What shoe should I use?

It is important that the orthosis does not shift within the shoe. A good shoe will help to achieve this. Choose a shoe where the opening can be adjusted with lacing or Velcro. Also make sure that the shoe has the correct heel height. The effective heel height should be 5/8" (1 cm). If the heel is too low, there is a risk for the knee to go into hyperextension.

### What shall I consider if it is a new patient?

Look carefully at the foot before and after the assessment and training. If any red marks appear, make sure that the padding is correctly positioned. If the patient has very sensitive skin, limit the training time.

### How do I apply the orthosis to the patient?

Place the orthosis in the patient's shoe. Then help the patient to put on the shoe with the orthosis simultaneously. When first placing the orthosis in the shoe, most patients (even those with limited use of one arm), can manage to put on the orthosis by themselves.

### How can I clean the orthosis between different patients?

The padding that is delivered with the orthosis can be machine washed in 104°F. The orthosis can be gently cleaned with mild soap and water.

Size chart

KiddieFLOW™ - NOT-FOR-RESALE	KiddieGAIT®- NOT-FOR-RESALE	KiddieROCKER®- NOT-FOR-RESALE	Size	Height	Foot plate
	283721008		Left, baby small	150 mm	110mm
	283721009		Left, baby medium	180 mm	125 mm
	283721010		Left, baby large	200 mm	140 mm
288941011	289001011	289021011	Left, small	220 mm	160 mm
288941012	289001012	289021012	Left, medium	257 mm	180 mm
288941013	289001013	289021013	Left, large	295 mm	200 mm
288941014	289001014	289021014	Left, xlarge	315 mm	210 mm
	283722008		Right, baby small	150 mm	110 mm
	283722009		Right, baby medium	180 mm	125 mm
	283722010		Right, baby large	200 mm	140 mm
288942011	289002011	289022011	Right, small	220 mm	160 mm
288942012	289002012	289022012	Right, medium	257 mm	180 mm
288942013	289002013	289022013	Right, large	295 mm	200 mm
288942014	289002014	289022014	Right, xlarge	315 mm	210 mm
288951011 (with D-ring)	289061011 (with D-ring)	289101011 (with D-ring)	Left, small	160 mm	220 mm
288951012 (with D-ring)	289061012 (with D-ring)	289101012 (with D-ring)	Left, medium	180 mm	257 mm
288951013 (with D-ring)	289061013 (with D-ring)	289101013 (with D-ring)	Left, Large	200 mm	295 mm
288951014 (with D-ring)	289061014 (with D-ring)	289101014 (with D-ring)	Left, xlarge	210 mm	315 mm
288952011 (with D-ring)	289062011 (with D-ring)	289102011 (with D-ring)	Right, small	160 mm	220 mm
288952012 (with D-ring)	289062012 (with D-ring)	289102012 (with D-ring)	Right, medium	180 mm	257 mm
288952013 (with D-ring)	289062013 (with D-ring)	289102013 (with D-ring)	Right, large	200 mm	295 mm
288952014 (with D-ring)	289062014 (with D-ring)	289102014 (with D-ring)	Right, xlarge	210 mm	315 mm

# Support for Better Life!

Camp Scandinavia AB, Karbingatan 38  
SE-254 67 Helsingborg, Sweden  
Phone: +46 42 25 27 01

allard|<sup>N</sup><sub>T</sub>

[www.allardint.com](http://www.allardint.com)

Tel +46 42 25 27 00  
Fax +46 42 25 27 25

**ALLARD INTERNATIONAL**  
Camp Scandinavia AB  
Karbingatan 38  
SE-254 67 Helsingborg SWEDEN  
[info@allardint.com](mailto:info@allardint.com)