PRIM

Novedad en ortopedia

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**ENG**  
Indications:  
The brace is indicated for patients requiring a recovery of the function of the foot lever and with limitation of the plantar flexion. It should be prescribed by a doctor. Parula paralysis or after an apoplexy.

### Cautions:

This brace should never be used for any other use different to the one prescribed. Do not allow the children to play with this device. When the patient to be treated is a child, the suitability of the brace state must be supervised periodically by a responsible adult. Some cautions must be followed when the patient has diminished physical or psychic abilities. Read these instructions carefully before using the brace. For any doubt, consult always your doctor or the establishment where you have acquired it before beginning to use it. Respect always these general instructions of use and the particular indications prescribed by your doctor, such as the complete duration of the treatment. The follow-on of the treatment should be frequently revised by your doctor. In case of secondary effects, inform your doctor. You should inform your doctor if you suffer of skin infections in the leg. Avoid the use of creams during the treatment in order to avoid the premature deterioration of the foams. Do not wear the brace when bathing or having a shower, etc. The user should be informed about the risk that the brace is overloaded in special situations (as jumping from a high height, etc) and can be broken, or when driving the brace could prevent a quick and sudden braking.

### Condition of use:

Cover the leg with a stocking or cotton sock. Use an appropriate footwear, of wide form, high reinforcements, of vamp and laces. Use the brace progressively. Use with a program of exercises controlled by the specialist. Do not use creams, lotions neither powders that soften the skin and prevent the perspiration. In case of secondary effects, allergy problems to the material, skin swelling due to its use, reddening due to pressures, bad blood circulation, dermatitis, skin injury due to perspiration, go to your doctor. Revisions at least every six months.

### Product maintenance conditions:

Do not expose to heat sources. Do not exercise external pressures on the brace. Clean with a wet cloth and dry it by hand. In case of breakage, go to the orthopaedist.

### \*Removable textile parts:

Wash at 40°. Do not use bleach. Do not tumble dry. Do not iron. Do not dry clean.

### Lasting of the product:

Theoretical lasting is of six months, depending on the conditions of its use, materials tiredness or morphological changes.

### Classification:

Class I medical device without a measuring function and supplied in a non-sterile condition. This orthosis complies with Council Directive 93/42/EEC, amended by 2007/47/EC, concerning medical devices.

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