Norsk

Bruksanvisning

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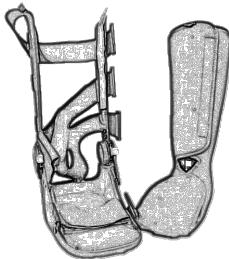
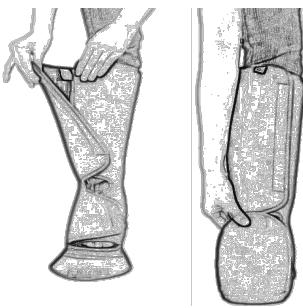
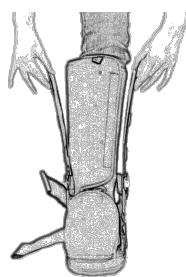
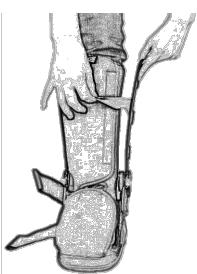
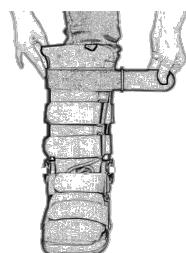
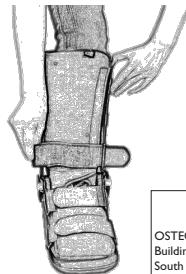
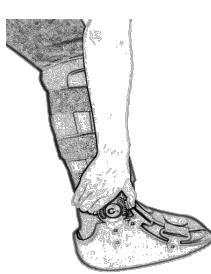
English

Instructions For Use

1. Unfasten all straps and remove liner from boot.
2. Place foot into the liner and make sure the heel is placed far back against the back portion of the liner. Fasten and wrap foot flaps first, then fasten the leg portion of the liner. The liner should be snug from top to bottom but should not be constrictive.
3. With the plastic still intact to the struts, gently open up both sides of the struts and place the foot into the boot with heel first then the forefoot.
4. Once the foot and leg is properly placed, remove the plastic and gently push the struts towards the liner to ensure the velcro on the boot touches the liner.
5. Re-attach the three straps on the lower leg section without securing them. Fasten the straps starting from the toe then moving up the limb. DO NOT over tighten as it may cause excess pressure on the injury resulting in discomfort.
6. To remove the boot, unfasten the straps, open the liner then remove from boot.

TO SET THE ROM JOINT

- A. Loosen the screw under the plastic cap on both sides. Do not remove the screw.
- B. Move the joint to desired position and re-position the pins.
- C. Only tighten the screw if the joint should be looked in a fixed position. This should only be done in 0°, 7,5°, 15° or 22°.
- D. Make sure the pins are mounted properly and that the opening of the plastic cover is turned away from the pins.

PRECAUTIONS

1. Patients without sensation need to be observed frequently for skin irritation, hot spots, etc. This product is restricted by Federal Law to be sold only under prescription or by a licensed health-care professional.
2. Always consult with your physician or therapist before making changes to the brace. This product is to be used under the supervision of a medical professional. This product is not intended for unsupervised public use.
3. Always inspect before use. Check for broken parts; torn, cut or frayed material; or buckles, or hook and loop fasteners that do not hold securely. Do not use damaged product. Discontinue use when torn, worn, frayed in any way.
4. Use caution when walking on slippery or wet surfaces to avoid injury.

WARNINGS

Do not use this product on patients unable to communicate pain or physical discomfort.

CLEANING INSTRUCTIONS

Wash with cold water, gentle cycle, mild soap. Do not dry clean or tumble wash. Do not use bleach. Dry in air current, and do not expose this brace to any heat source.

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