OPED GmbH
Medizinpark 1
83626 Valley/Oberlandern
Germany
Fon +49 (0)8024/60818-210
Fax +49 (0)8024/60818-299
mail@oped.de
www.oped.de

OPED AG
Hinterbergstrasse 26
6312 Steinhauen
Switzerland
Fon +41 (0)41/748 53 35
contact@oped.ch
www.oped.ch

OPED UK Ltd.
Unit 5
Avro Business Centre
Avro Way
Bowerhill
Melksham
SN12 6TP
United Kingdom
Phone +44 (0)1380 722177
enquiries@oped.biz
www.oped-uk.com

Orthotic & Prosthetic Centre Pty. Ltd.
OPC Health an OPED company
26-32 Clayton Rd
Clayton VIC 3168
Australia
Phone +61 (0)1300 672 937
sales@opchealth.com.au
www.opchealth.com.au

OPED Medical, Inc.
444 Tom White Road
Braselton, GA 30517
USA
Phone +1 (770) 945 - 0150
Fax +1 (770) 945 - 0370
info@opedmedical.com
www.opedmedical.com

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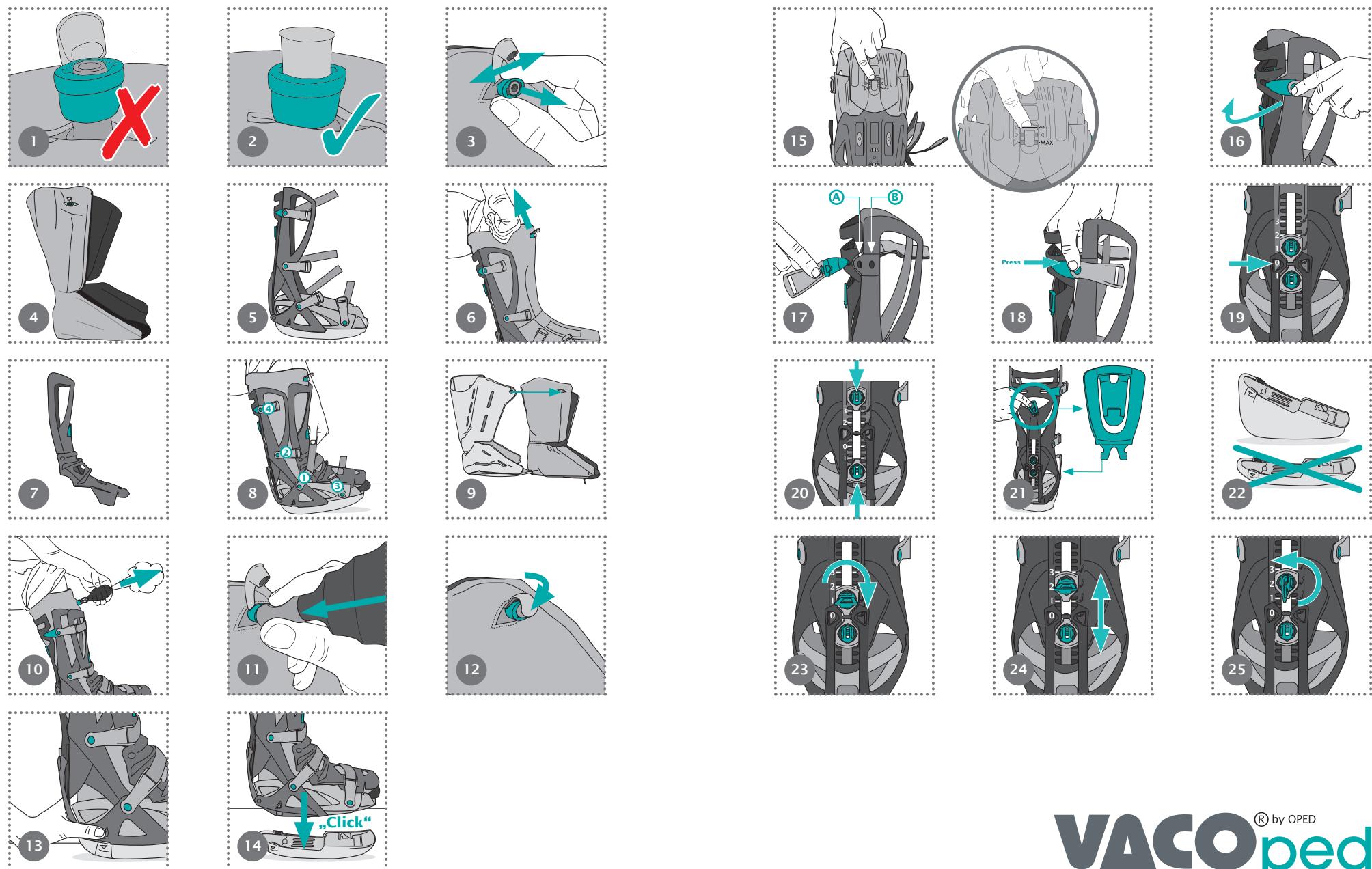
VACOped

Please read the instructions for use and safety advice before using VACOped.

Purpose: Lower leg and foot orthosis for immobilisation of the joints in defined, adjustable positions, and for mobilisation in definable ranges of movement. You will find a readable instruction manual on our OPED homepage.

Safety information

- ⚠** The product is intended only for use on a single patient.
- ⚠** VACOped is not designed for dynamic stresses such as jumping and fast walking. If you fall with the VACOped fitted, check your VACOped immediately for damage. If the VACOped is damaged, it can impair your safety.
- ⚠** Never wear the product over open wounds and only wear it after prior medical instruction.
- ⚠** The patient may only change the position of the device after consultation with the physician responsible for their care.
- ⚠** If excessive pain, numbness or sensation changes occur, or if unusual reactions or an unpleasant feeling are experienced while wearing the device, please contact your doctor or orthopaedic technician without delay.
- ⚠** This medical product has been developed for the indications listed and can be damaged by unintended or inappropriate use, by activities such as sport, falls or other accidents, and by liquids that are not compatible with plastics.
- ⚠** When used in combination with another medical product, consult your doctor.
- ⚠** Only spare and/or replacement parts approved by OPED GmbH should be used.
- ⚠** Please note the Road Traffic Act applicable in your country.
- ⚠** Note: Not made from natural rubber latex. 
- ⚠** Check that the sole fits securely - otherwise there is a risk of slipping!
Sole: When the toe section is fully extended, the gait may be modified or restricted.
- ⚠** The small spheres in the vacuum cushion can slide around. We therefore recommend opening the valve every two to three days, smoothing any collections of small spheres out and then restoring the vacuum.
- ⚠** Storage: The valve flap must always be closed! The valve ring must be pushed downwards and the valve flap closed (Figure 1 + Figure 2).
- ⚠** This product is intended to be used for no more than 4 months.
- ⚠** Note: When wearing the orthosis, make sure that the shoes are high enough on the opposite side or that the sole is raised to compensate for any difference in height.

How to use (instruction video can be found at app.oped.de or app.oped.ch)

Fitting the VACOped

Pull the valve ring upwards (details can be found in "Valve functions") to allow air to flow in (Figure 3). If the small spheres in the liner (Figure 4) have grouped together, stroke them smooth. Place your foot in the fabric liner (Figure 4) and position it with the fabric liner all the way back in the lower shell (Figure 5). Then close the fabric liner. Pull the liner slightly upwards (Figure 6). Fit the upper shell (Figure 7) (top: outer towards the lower shell; bottom: tabs into the lower shell) and close the strap by hand and as the pain allows (see graphic for sequence) (Figure 8). Retighten the straps after a few steps. When fitting the upper shell, ensure that the valve is not kinked. The upper shell should not be positioned against or over the valve. If the physician permits, apply some pressure briefly so that the foot is moulded in the cushion. Then create the vacuum.

Adjusting and securing the vacuum cushion

Readjustable vacuum cushion: Inside the liner, there is a cushion containing thousands of small spheres (Figure 9) which adapt perfectly to the anatomy and any swelling of your ankle joint. Pressure-sensitive areas and swellings can be relieved through the targeted redistribution of these small spheres. To ensure that the spheres do not slide around inside the cushion, the air is extracted via the valve using a suction pump, creating a vacuum (Figure 10).

Valve functions

Open the valve / softens the vacuum cushion:

Pull the valve ring upwards and move it to the side to allow air to flow in (Figure 3). Then slide the valve ring back to its original position.

Close the vacuum / hardens the vacuum cushion:

Push the valve ring downwards. Attach the suction pump and squeeze it (approx. 7-8 times) until it no longer inflates. Squeezing the pump removes air from the cushion and creates a vacuum. This fixes the adapted fit in place (Figure 11). To finish, close the valve cap (Figure 12).

Closing the valve cap

Push the valve ring downwards and close the valve cap. This sets the vacuum (Figure 12).

Changing the cushion liner

Note: The replacement liner does not contain a cushion! When changing the liner, the cushion must be removed from the used liner and inserted into the new one!

Open the rear zip fastener all the way and remove the vacuum cushion from the used liner. Open the valve and shake out the cushion so that the small spheres can redistribute themselves (see Valve functions). Establish the vacuum (see Valve functions), place the vacuum cushion in the new liner and slide the valve through the button hole (Figure 9).

A second cover is not included in the box in all countries. If the supplied package does not include a second cover, this must be ordered separately from OPED.

VACOped

How to use

Angle adjustment

Delivery condition: Neutral zero position ("0" \pm 0°) (Figure 19)

There are two possible uses:

1st option: Fixed ankle (fixed setting): The upper and lower adapter screws are not separated (Figure 19).

2nd option: Limited freedom of movement (ROM setting): For use as soon as movement is permitted (Figure 20). The distance between the adapter screws determines the extent of the ankle joint's range of movement (ROM).

The range of movement can be changed in 5° increments from 15° of dorsal extension (lifting of the toes) to 30° of plantar flexion (equinus position = toes inclined).

Note: The use of a range of movement (ROM) is the decision of the doctor / physiotherapist.

Scale on the product	Corresponds in degrees	Ranges of movement (ROM)
3	+30°	
.	+25°	
2	+20°	Plantar flexion, starting from the neutral - zero position
.	+15°	
1	+10°	
.	+5°	
0	0°	Neutral - zero position
.	-5°	
1	-10°	Dorsal extension, starting from the neutral - zero position
.	-15°	

Changing the angle setting

Upper adapter screw: End stop for plantar flexion (equinus position = toes inclined)

Lower adapter screw: End stop for dorsal extension (lifting of the toes)

Remove the key from the holder (Figure 21) and insert it into the screw. Turn the key horizontally (Figure 23). Lift the screw and push it into the desired position (Figure 24). Turn the key vertically (Figure 25). Repeat the steps above if necessary with the other screw.

Note on treating Achilles tendon injuries in the equinus position: With a fixed angle setting between +15° and +30°, please use the wedge sole (Figure 22). If the wedge sole isn't included in the package, it has to be ordered separately from OPED.

Fitting and changing the sole

Removing the sole

Push both unlocking buttons simultaneously (Figure 13).

Fitting the sole

Position the VACOped straight and centred on the sole before pushing it against the sole (you will hear two clicks) (Figure 14).

Adjusting the sole length

Remove the sole (see Fitting the sole) and push the unlocking button. Pull the toe section to the desired length (Figure 15).

Adjusting the calf circumference

Push the buckle out of the lower shell (Figure 16). Push the buckle to the desired position. Position A = large calf circumference, Position B = small calf circumference (Figure 17). Push the buckle into the required hole (Figure 18).

Repeat the process on the opposite side.

Patient information

Pressure points / problem-solving

Remove the orthosis from your foot. Pull the valve ring to allow air to flow into the cushion. Spread the cushion filling evenly at the pressure point. Re-fit the orthosis (see How to use). Using a finger, check that there are enough small spheres in the cushion between the pressure point and the outer lower shell.

Showering / washing

We recommend using a shower protector when showering. If the orthosis or its cover gets wet, they must be dried.

Sleeping

Before you go to sleep, place a towel, such as a hand towel, underneath the device so that the bed sheets are not damaged.

VACOped

Product characteristics

Sizes

The product can be worn on the left or right.

Recommendation: At the size interfaces between the Small/Medium and Medium/Large versions, supply the smaller VACOped size if possible.

	Small	Medium	Large
EUR	35 – 40	40 – 45	45 – 50
UK	2 ½ – 6 ½	6 ½ – 10 ½	10 ½ – 14
US (M)	3 ½ – 7 ½	7 ½ – 11 ½	11 ½ – 15
US (F)	4 ½ – 8 ½	8 ½ – 12 ½	12 ½ – 16

Indications

- Ankle joint fractures
- Metatarsal fractures
- Achilles tendon ruptures
- Calcaneal fractures
- Severe sprains
- Prostheses
- Arthrodeses

Contraindications

None known

Information regarding care / cleaning / disinfection / sterilisation

You are receiving this medical product in a hygienically perfect condition.

Cleaning instructions for patients:

- Lower shell: Dishwasher at 60 °C
- Strap: Washing machine at 40 °C
- Terry liners: Washing machine at 60 °C

Remove the vacuum cushion from the Terry liner before washing it. Close the zip and Velcro fasteners again for washing.

- Drying: Air dry, do not dry in a tumble dryer.
- Disinfection: Wipe disinfection with plastic-compatible disinfectant.
- Sterilisation: This medical product is not suitable for sterilisation.

Technical data / parameters

Materials used:

Hard lower shells (PP/POM); Liners (cotton/PU foam); Cushion sleeve (PVC); Filler material (EPS spheres); Strap (PP fabric); Sole (EVA); Upper shell (PP/PU). All parts and components are latex-free.
Note: Not made from natural rubber latex.

Transport conditions / storage conditions / disposal

Storage: Store in a dry and hygienic place and protect from direct sunlight

Germany/Switzerland: The medical device is a rental product and must be returned to OPED!

Other countries: Please dispose of the medical device in residual or household waste.

Warranty

The warranty is provided in line with statutory regulations.

In the event of complaints relating to the product, such as damage or problems with the fit, please contact the manufacturer directly. Only serious events which can lead to a significant deterioration in health or death should be reported to the manufacturer and the responsible authorities of the member state. Serious events are defined in Article 2 (65) of Directive (EU) 2017/745 (MDR).

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