allard

Support for better life!

Instructions for Use Vission Fingersplint

Art. No. 46020

46020 Vission Finger Splint

Carefully read this instruction sheet before use!

EN

Intended Use/Indications

For immobilizing the fingers following trauma or operation and preventing contractures of the finger joints. Can also be used in conservative finger therapy and Dupuytren syndrome. Conditions requiring temporary finger immobilization. Single Patient-multiple use.

Contraindications

No known.

Adjustments

This product should only be altered by medical professional person.

Warnings

Make sure that the straps are not too tight. If any sign of skin irritation occurs, immediately remove the support. Check your skin on a daily basis.

Material Specifications

This product is made from Polyamid, Spandex and Polyurethane. Splint is made in aluminium. This product is not flame retardant. Does not contain Latex.

Washing Instruction

Close all Velcro prior to laundering. Remove spoons before laundering. Machine wash in 40°C (104°F) gentle cycle. Use a washing bag. Dry flat. Do not tumble dry or dry clean. Take notice of how the splint is positioned in the pocket to ensure correct position after wash.

Disposal


Disposal in accordance with local legislation.

Information

Any serious incident that may occur in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and / or patient is established.

Who to Contact

If any problems occur, remove the support and contact your fitter or store where purchased.

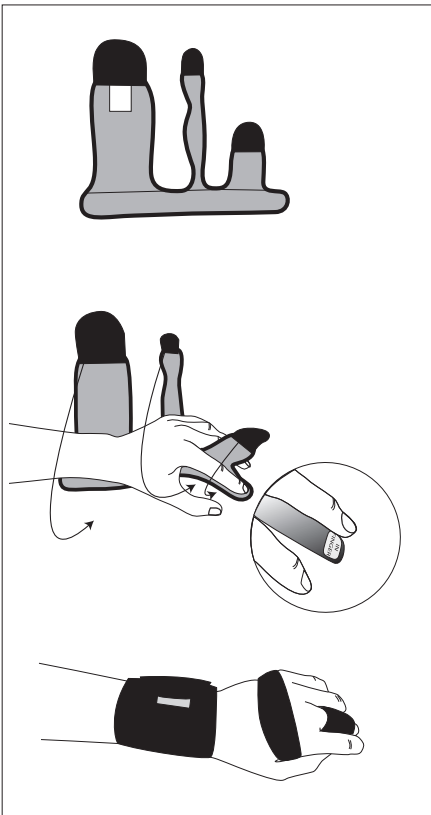Instructions for use

Check that the product is complete and without damage.

1. Open all Velcro straps. Product should be open and flat.
2. Position hand and fingers on the support. The treated fingers should be positioned over the long extended finger section.
3. Secure the strap around the treated fingers on the long extended finger section.
4. Wrap the wider strap around the wrist and secure.
5. Attach the middle strap around the mid-section of the palm inside of the thumb but around all fingers.
6. Adjust for comfort.

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