allard

Support for better life!

Instructions for Use

Selection[®]
Multiopen
Art. No. 35209

Instructions For Use

EN	SELECTION® MULTIOPEN	p. 4
SV	SELECTION® MULTIOPEN	p. 5
DA	SELECTION® MULTIOPEN	p. 6
NO	SELECTION® MULTIOPEN	p. 7
FI	SELECTION® MULTIOPEN	p. 8
IT	SELECTION® MULTIOPEN	p. 9
NL	SELECTION® MULTIOPEN	p. 10
DE	SELECTION® MULTIOPEN	p. 11
ES	SELECTION® MULTIOPEN	p. 12

Intended use

Intended to off-load and support the wrist to a neutral resting position to maintain important wrist functions. For one persons use only.

Indications:

Post healed distal radius fracture, Rheumatic disorders, Soft tissue injuries, work related strain injuries or with degenerative changes. Single Patient-multiple use.

Contraindications

Not to be used for non-healed fractures.

Adjustments

This product should only be altered by medical professional person.

Warnings

Make sure that the straps are not too tight. If any sign of skin irritation occurs, immediately remove the support. Do not use product when wet or on wet skin. Check your skin on a daily basis.

Material Specifications

This product is made from polyester, nylon and elastan. Spoon is made from aluminum. This product is not flame retardant. Latex free.

Washing Instruction

Close all Velcro prior to laundering. Remove spoons before laundering. Machine wash in 40°C (104°F) gentle cycle. Use a washing bag. Dry flat. Do not tumble dry or dry clean.

Disposal

Disposal in accordance with local legislation

Information


Any serious incident that may occur in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and / or patient is established.

Who to Contact

If any problems occur, remove the support and contact your fitter or store where purchased.

Instructions for use

1. Check that the support is complete and without damage.
2. Open the lacing and Velcro straps. Position the palm so that the stay rests comfortably on the inside.
3. Gently fold over the Velcro part so that the orthosis fits loosely around the arm.
4. Close the thumb strap over the dorsal side of the hand.
5. Then tighten the lacing.
6. Adjust for comfort.

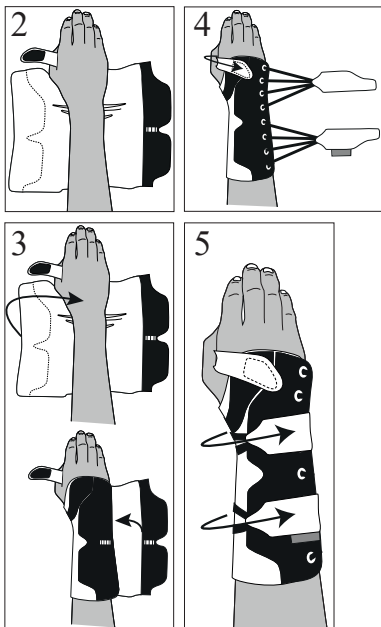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