CTS 46050

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Indications / Intended Use

This product is intended to prevent numbness in the hands at night due to Carpal Tunnel Syndrome. The splint will keep the wrist in a neutral position and prevent bending of the wrist during night.

Contraindications

No known.

Adjustments

This product should only be altered by a medical professional.

Warnings

Make sure that the straps are not too tight. If any sign of skin irritation occurs, immediately remove the support. Check your skin on a daily basis.

Material Specifications

This product is made from Boltaron material. This product is not flame retardant. Latex free.




Washing Instruction

The splint can be hand washed with mild soap. Rinse carefully. Straps can be machine washed in 30°C (86°F). Dry flat. Do not tumble.

Who to Contact

If any problems occur, remove the support and contact your fitter or store where purchased.

Instruction for use

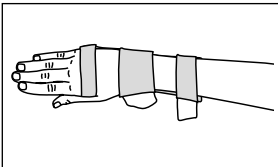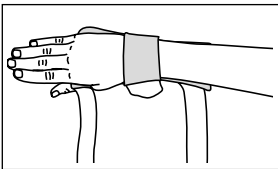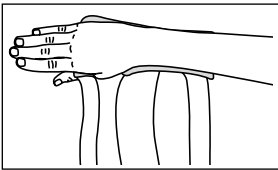Check that the support is complete and without damage.

1. Place the hand into the splint.
2. Start by wrapping the middle strap around the wrist.
3. Then attach the strap over the knuckles.
4. Finally attach the upper strap.
5. Adjust all straps for comfort.

If the length of the strap is shortened, please heat seal the adjusted end to avoid fraying.

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Camp Scandinavia AB, Karbingatan 38