# **WristLacer II**

**28578**

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#### Indikationer

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#### Intended use/Indications

Post-traumatic, post-operative, carpal tunnel syndrome, rheumatic disorders and spasticity.

#### Contraindications

None known.

#### Adjustments

This product should only be altered by medical professional person.

#### Warnings

Make sure that the straps are not too tight. If any sign of skin irritation occurs, immediately remove the support. Check your skin on a daily basis.

#### Material specifications

This product is made from perforated suede imitation which allows moisture to evaporate from the skin and with a felt lining on the inside. Polypropylene, polyester, nylon, aluminum rails, vinyl, cotton, brass.

#### Washing Instruction





Close all Velcro prior to laundering. Machine wash in 40°C (104°F) gentle cycle. Use a washing bag. Dry flat. Do not tumble dry or dry clean.

#### Who to Contact

If any problems occur, remove the support and contact your fitter or store where purchased.

## Instructions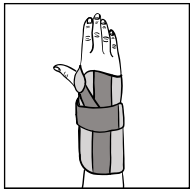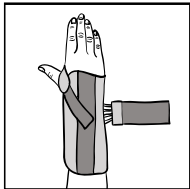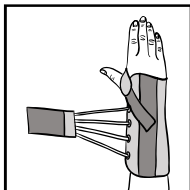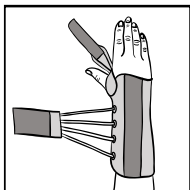

1. Check that the product is complete and without damage.
2. Open all Velcro straps.
3. Pull the orthosis on like a glove. The wide stay should be positioned inside the hand (the volar side). The more narrow stay should be on the upper side of the hand (the dorsal side).
4. Attach the thumb strap.
5. Then pull the strap holding the lace in a straight line around the wrist.
6. Check that the orthosis is comfortable. Check that the stay inside the hand (volar side) follows the contours of the wrist in a neutral position (if not, adjust the stay). If needed remove the orthosis and adjust the stays on both sides. Put back on again following steps 3-5 above.

Med Spec Item No. Left	Med Spec Item No. Right	Size	CM
223329	223339	XX-Small	<12 cm
223321	223331	X-Small	12-14 cm
223322	223332	Small	14-16 cm
223324	223334	Medium	16-18 cm
223325	223335	Large	18-20 cm
223326	223336	X-Large	20-22 cm
223327	223337	XX-Large	22-25 cm

Medical Specialties, Inc.  
 4600-K Lebanon Rd.  
 Charlotte, NC 28227  
 p: 800-334-4143 f: 800-694-9060  
 p: 704-694-2434 f: 704-694-9060  
 e-mail: request@medspec.com  
 www.medspec.com

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MDSS GmbH
Camp Scandinavia AB, Karbingatan 38