

# CMC-X™ & CMC-X™ Lacer

## Thumb Stabilizer


### Instructions For Use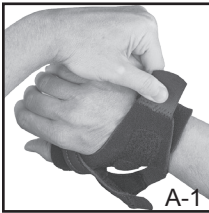

A-1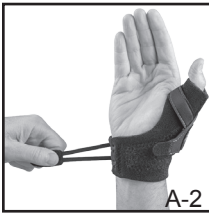


A-2

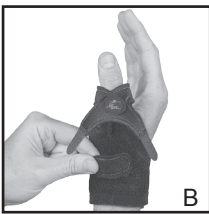B

C

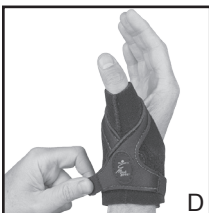D

Medical Specialties, Inc.  
4600-K Lebanon Rd.  
Charlotte, NC 28227 USA  
p: 800-334-4143 f: 800-694-9060  
p: 704-694-2434 f: 704-694-9060  
e-mail: request@medspec.com  
www.medspec.com

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### English

Before using this product, please carefully read all instructions and warnings.

**Warning:** If you experience any increased pain, swelling, or any unusual reactions while using this product, immediately consult your medical professional. This product is for single patient use only. There is no guarantee that use of this product will prevent injury.

A-1. Disengage "X" strap. Wrap body and pull outer hook & loop closure to secure around wrist.

A-2. Disengage "X" strap and pull lace closure tight to secure around wrist.

B. Disengage stay pod and reposition at base of thumb over CMC joint.

C. Relax thumb while pulling "X" strap and secure near base of thumb.

D. Relax thumb while pulling and securing ends of "X" strap.

**Washing Instructions:** Hand wash in warm water with mild detergent and no bleach. Rinse thoroughly. Air dry.

**Warranty:** Products manufactured by Med Spec have a limited warranty against defects in materials or workmanship for a period of six months from date of shipment. Please contact the business that supplied you with this product if you wish to return it.

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