BASIC THUMB SHORT & LONG

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Indikationer

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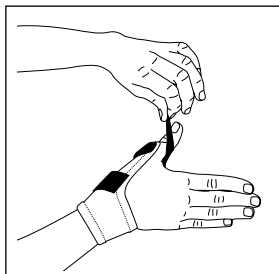
ENGLISH

27443 & 27444 BASIC THUMB ORTHOSIS

Carefully read this instruction sheet before use!

Indications / Intended Use

This product is intended to be used for instability in the thumb joints (CMC- or MCP joints), inflammation, joint stress and for rheumatic degenerations.

Contraindications

No known.

Adjustments

This product should only be altered by a medical professional person.

Warnings

Make sure that the straps are not too tight. If any sign of skin irritation occurs, immediately remove the support. Check you skin on a daily basis..

Material Specifications

This product is made from: 74% Cotton, 19% Nylon and 7% Lycra with an plastic support. Shape the plastic support carefully to fit the shape of the thumb. Avoid repetitive bending. This product does not contain Latex, and is not flame retardant.

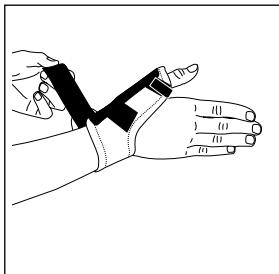
Washing Instruction

Close all Velcro prior to laundering. Machine wash in 40°C (104°F). Use a washing bag. Dry flat. Do not tumble dry or dry clean.

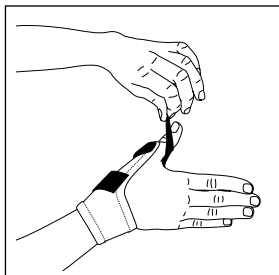
Who to Contact

If any problems occur, remove the support and contact your fitter or store where purchased.

Instruction for use

1. Check that the support is complete and without damage.
2. Open the Velcro for the wrist.
3. Place the orthosis over the thumb.
4. Pull around the wrist and secure the Velcro.
5. Adjust for comfort.

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Camp Scandinavia AB, Karbingatan 38