GN0510 - Genuranger Universal

 **MEDICAL DEVICE**

 **SINGLE PATIENT - MULTIPLE USE**

 **CAUTION**


 Promedics Orthopaedics Ltd.
Port Glasgow Industrial Estate, Gareloch Road, Port Glasgow,
Scotland, UK, PA14 5XH
Tel: +44 (0) 1475 746 400 | Email: enquiries@promedics.co.uk
www.promedics.co.uk

   **EMERGO EUROPE**
Prinsessegracht 20
2514 AP The Hague
The Netherlands

©2021 Promedics Orthopaedics Ltd. All Rights Reserved.

GN0510FUB revised 06/21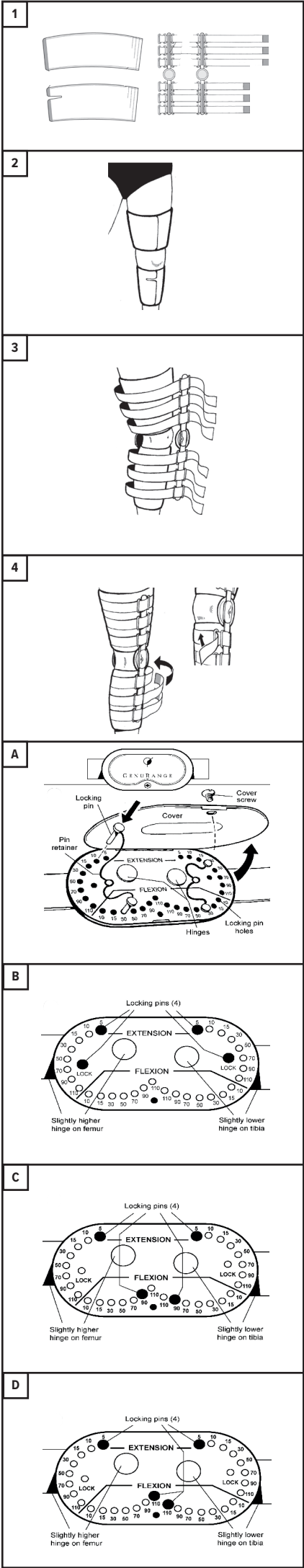

ENGLISH

Product Description

A post-op knee brace to provide controlled range of motion at the knee joint. One size universal wraps can be cut to fit most sizes.

Usage Information (See diagram)

- Open the straps (attached to thigh hinges) and detach the thigh cuff and the calf cuff. The cuffs may be cut shorter if required as shown above.
- Apply the thigh cuff to the thigh and the calf cuff to the calf.
- Set the Genurange hinges to lock, limited flexion/extension of full flexion/extension. (See 'Setting the Genurange Hinge' instructions below). With padded side towards each side of the knee, attach the hinges – with straps attached – to the thigh cuff and calf cuff. Ensure that the hinge flexes at the point of knee flexion and in the correct direction.
- Close all straps firmly but comfortably by passing through buckles and pressing hook/loop closure.

Setting the Genurange Hinge:

- To open Genurange Hinge: Use the Philips screwdriver provided, open the cover of the Genurange Hinge. Retain the Philips screw in a safe place.
- To limit flexion/extension: Place the 2 flexion pins in eg. 90 degrees to limit flexion. Place 2 extension pins in eg. 5 degrees to limit extension.
- To lock the Genurange hinge: Place 2 of the small removable locking pins in the "LOCK" position. Place the 2 remaining pins in any position (eg 5 degrees Extension): These may be used later to set extension.
- To set the Genurange hinge in full flexion/extension allowing full range of movement: Place the 2 extension pins to 5 degrees for greatest extension. Place the 2 flexion pins in 110 degrees for greatest flexion.

Sizes and Fit

Size	Length
XS	40cm – 16"
S	45cm – 18"
M	50cm – 20"
L	55cm – 22"
XL	60cm – 24"

Indications for Use

- Following surgery of the knee joint
- For immobilisation after knee joint injuries and subsequent mobilisation
- Knee ligament injuries

Contraindications

- Do not use over open wounds
- Any known allergy and/or hypersensitivity to any material listed in product composition

Warnings and Precautions

- We recommend the initial fitting of this device be conducted by a suitably qualified healthcare professional who will advise the period of use.
- Carefully read all instructions and warnings prior to use.
 - Follow all instructions to ensure proper performance of the device.
 - Do not use if liniments, ointments, gels, creams or any other substances have been applied to the affected area.
 - Do not re-use for another patient, doing so risks cross infection and can compromise product integrity.
 - Perform regular skin and circulation checks, especially patients with diabetes, vascular deficiencies and neurological conditions.
 - Should any adverse reactions occur, please cease use and contact your healthcare professional or provider of this product.
 - The durability of the device may be compromised by certain factors, e.g., objects with sharp edges or damage to the hook and loop fasteners. To prevent this the hook and loop fasteners should always be fastened when the device is not being worn or when being washed.

Washing and Care Instructions

- Close over all straps/cuffs.
- Hand wash in cool water using mild detergent.
- Rinse and dry flat.
- Do not spin or tumble dry.
- Do not use fabric conditioner at any time.
- Check for any degradation that may affect use.
- Periodically check cuff/strap fasteners are operating securely.

Product Composition

Nylon, Polyester, Polyamide, Polyurethane, Aluminium, Steel, Elastane.

Storage and Transport Conditions

Store in a cool, dry place out of direct sunlight, in the original packaging.

Recycling and Disposal

Packaging and constituent parts should be recycled or safely disposed in accordance with local or national laws.

Serious Incident

Report any serious incident to the manufacturer and the competent authority of the EU Member state, or country in which you reside.

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