

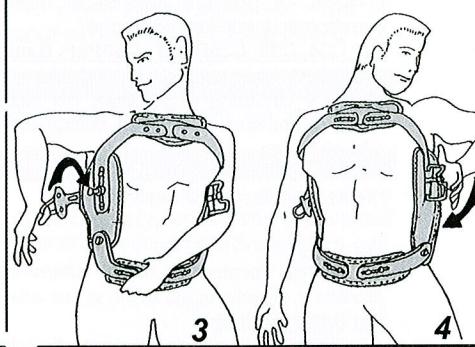
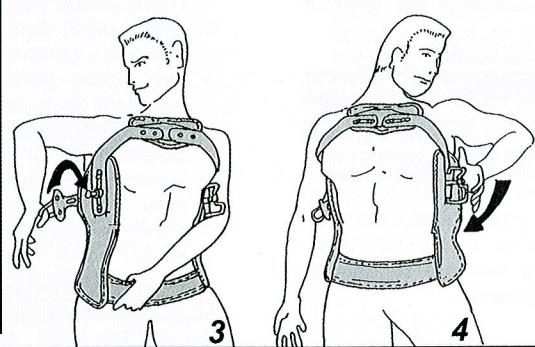
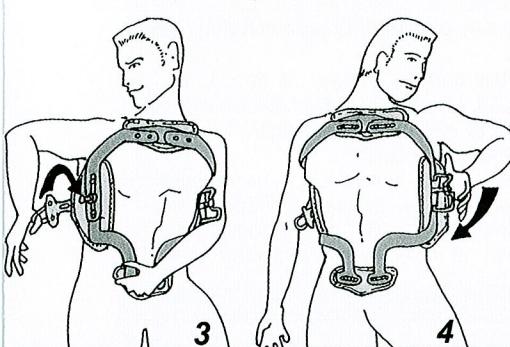
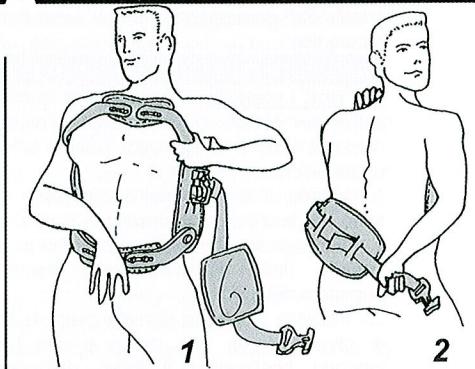
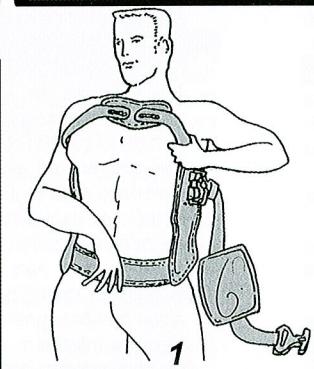
C34

C35, C35PlusF, C35Plus Basic


C37, C37A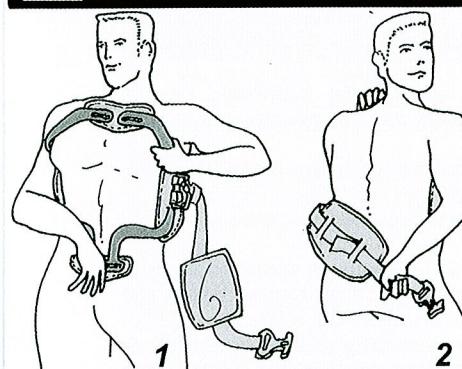

C38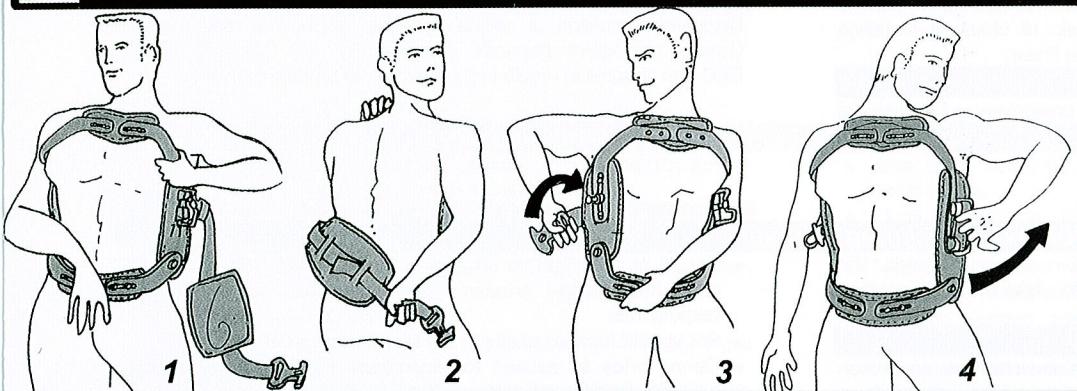



C35PlusF

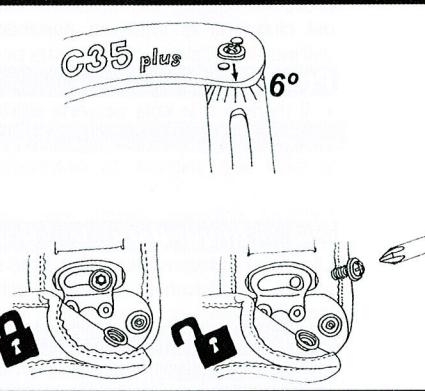Detalles / Details / Détails / Detalhes / Dettagli / Detaljer / Detaljer / Yksityiskohdat / Detaljer / Details / Details

C34, C35, C35PlusF, C35Plus Basic, C37, C37A

C38

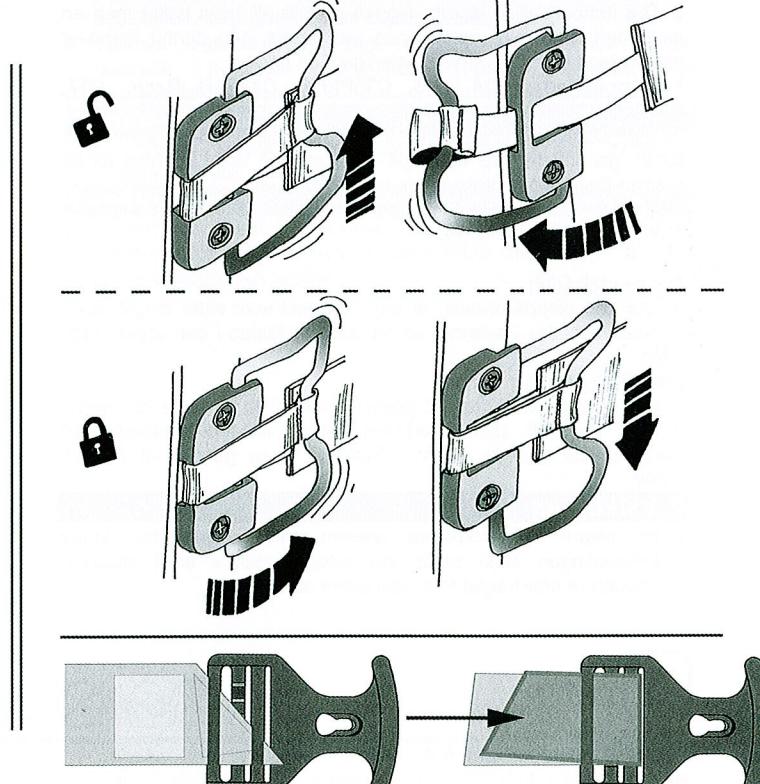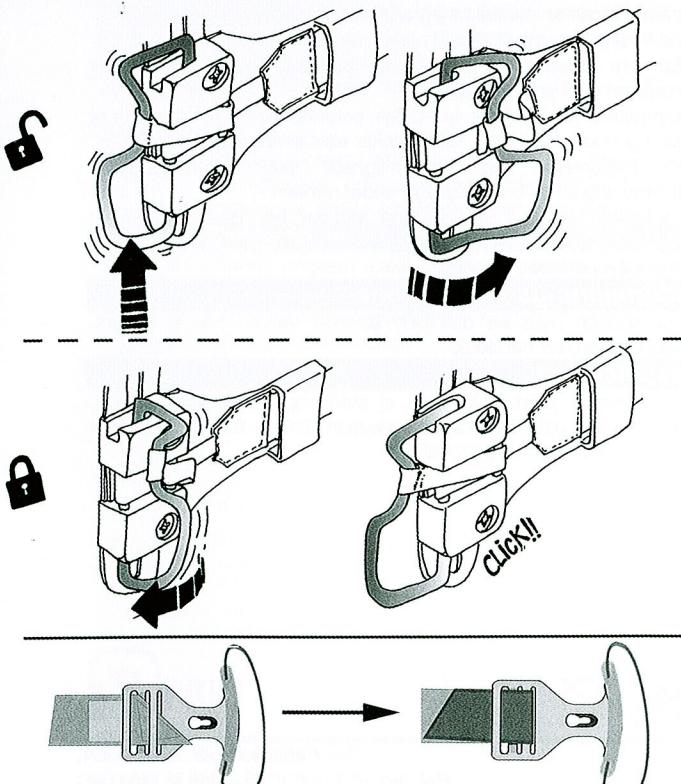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## ENG - Instructions of Use for Hyperextension Braces

Read carefully these instructions before using the orthosis. For any doubt, consult always the doctor or the orthopaedic technician.

### Fitting instructions

**For all the models:** The positioning of this brace by the patient may need the help of another person. It is advisable to put on the brace in front of a mirror.

Make sure that the tensioning closure system is on its open position.

- 1.- Position the brace onto the patient in accordance with the direction of the arrow stuck on the upper part of the brace.
- 2.- Pass the lumbar plate through the back holding the belt with one hand and the brace placed over the patient with the other. The help of a second person is advisable to achieve this operation.

**For models C34, C35, C35PlusF, C35Plus Basic, C37, C37A:**

- 3.- Without releasing the brace, introduce the thumb of the hand which is holding the belt through the blue cord of the plastic closure buckle, pulling, hook the buckle in its housing.

**NOTE C35PlusF:** The tilting of the sternal support can be regulated (moderate tilting 6° by 6°).

### For model C38:

- 3.- Without releasing the brace, hold the plastic closure buckle, pulling, hook the buckle in its housing.

### For all the models:

- 4.-Finish to correctly position the brace with both hands and proceed to the tensioning of the brace by turning the metallic ring to its final position.

### Cautions

- The doctor is the person qualified to prescribe the orthosis and decide the duration of the treatment. The orthopaedic technician is the qualified person to shape the plates and

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prepare the brace according to the required contour and to fit the brace.

- For a correct effectiveness, you should respect the daily time of use following the doctor or your orthopaedic indications.
- For a correct function, you should use the correct size of the orthosis.
- Orthosis of personal and untransferable use. It should not be used for the use of another patient.
- This orthosis should never be used for any other use different to the one prescribed. Do not allow children to play with this device.
- You should inform your doctor if you suffer of some skin affections. Avoid the use of creams, in contact with the orthosis, during the treatment.
- On the pressure points, the skin should not be injured nor irritated.
- Do not wear the orthosis when bathing or having a shower, etc. with the exception of model C38 which can be worn.
- Check periodically that the belt and/or tensionings are tight and in good conditions (without grease, cuts, rips, etc.).
- For hygienic and comfort reasons, it is advisable to wear a cotton garment underneath your back support.
- Check periodically that the screws don't have lose their initial torsion, if it is so, please contact your doctor or orthopaedic technician to adjust them.

### Maintenance instructions

Wash with a cloth, lukewarm water. Do not bleach. Dry with a dry cloth.

### Classification

Medical Device, Class I, non sterile, with no measurement function. This product fulfils the Medical Device Directive 93/42/EEC, by which the medical products are regulated.

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