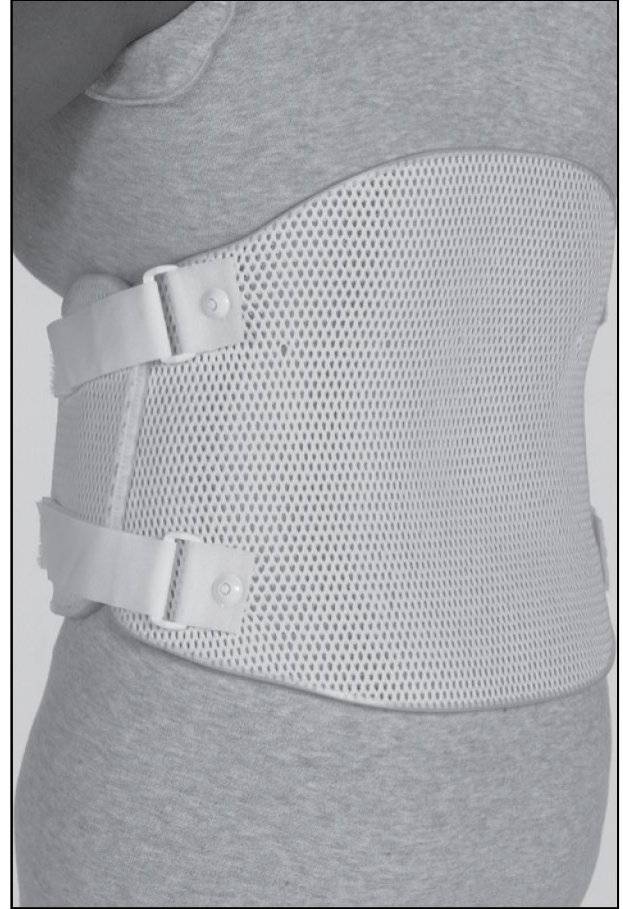
# 3D-LITE™

## CUT-4-CUSTOM

### TLSO

### LSO

**CS** Návod k použití  
**DA** Brugsanvisning  
**DE** Bedienungsanleitung  
**EL** Οδηγίες χρήσης  
**EN** Instruction for use  
**ES** Instrucciones de uso  
**FI** Käyttöohje  
**FR** Mode d'emploi  
**HU** Használati utasítás

**IT** Istruzioni per l'uso  
**LT** Naudojimo instrukcija  
**NL** Gebruiksaanwijzing  
**NO** Bruksanvisning  
**PL** Instrukcja użytkowania  
**PT** Instrução de uso  
**RO** Instrucțiuni de folosire  
**SK** Návod na použitie  
**SV** Instruktion för användning

**kinetec**®

[www.kinetecinternational.com](http://www.kinetecinternational.com)

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**Intended use**

Cut-4-Custom is intended to stabilize the spine. The orthosis is available in different heights and full measurements can be found in the sizing table (separate document).

**Indications**

Compression fractures; Post-operative use; Stabilization of the spine.

**Contraindications**

Instable spine; Instable fractures.

**Precautions & Warnings**

- ◆ Do not pull the straps too tight, as this can cause poor circulation and/or skin irritation.
- ◆ Check skin on a daily basis.
- ◆ If you see any skin discoloration or feel any irritation or other problems, take it off immediately and contact your fitter/subscriber.
- ◆ Avoid warm temperatures and strong sunshine.
- ◆ Store in a dry, dark place at temperatures between 0°C and +40°C.
- ◆ The orthosis is sensitive to chlorinated water.
- ◆ The orthosis can be used multiple times on the same patient.
- ◆ Dispose in accordance with local regulations. Can be handled with the household waste.
- ◆ Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

**Adjustment**

This product should only be altered by a medical professional person. The fitter should during the fitting process make sure that sensitive areas such as the arm pits and groins are free from pressure, as this can cause numbness and discomfort for the user.

**Instructions for use**

- ◆ Check that the product is complete and without damage.
- ◆ The front and the back sections should be attached together on one side.
- ◆ Position the back section on a bed so you will be able to lay your back into it (picture A).
- ◆ Move the front section over the stomach into its proper position. Check that legs can be bent without interference (picture B).
- ◆ Secure the straps on the open side and adjust.
- ◆ Loosen all the straps without removing them from the D-ring and adjust so the orthosis is in position and is comfortable.
- ◆ Rise gently to the edge of the bed (you might need assistance to get up) (picture C). Check that the orthosis is comfortable, adjust if necessary.

**Who to contact**

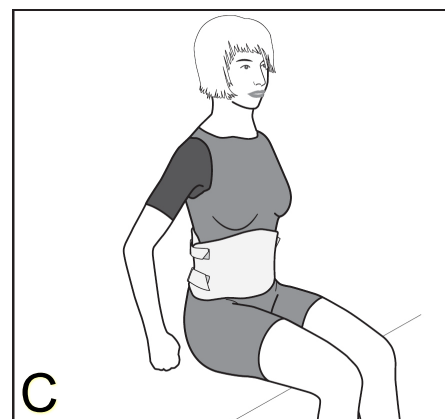
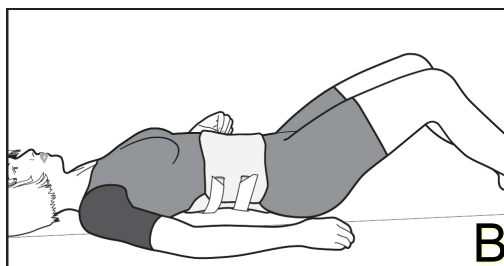
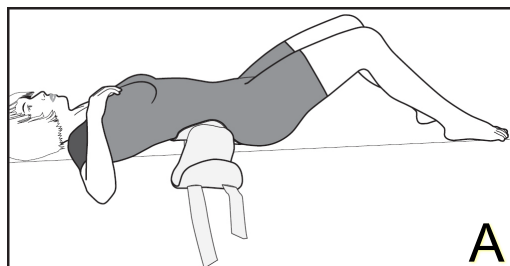
If any problems occur, please contact your orthotist or fitter.

**Material specifications**

Rigid section: Polyester weave impregnated with non-toxic resin.  
This product does not contain Latex.

**Cleaning instructions**

The rigid orthosis should be cleaned with a damp cloth soaked in mild soap. Rinse with clean water and allow to air dry.

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